## **Inhaltsverzeichnis / Contents:**

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*Please fix the skai-padding with velcro as follows.*

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*Please fix the skai-padding with the cords as follows.*

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
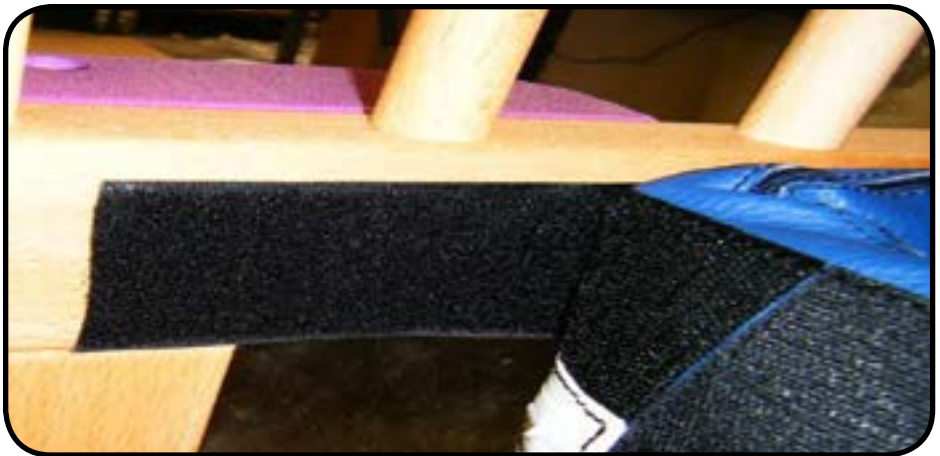
*Please fix the skai-padding with velcro and sticky back as follows.*

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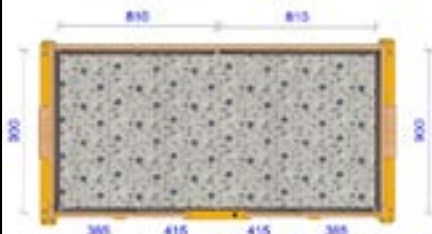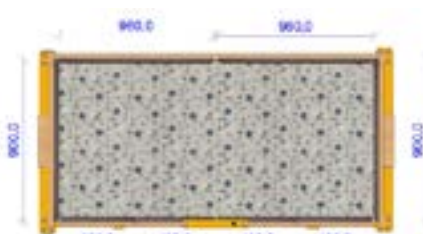
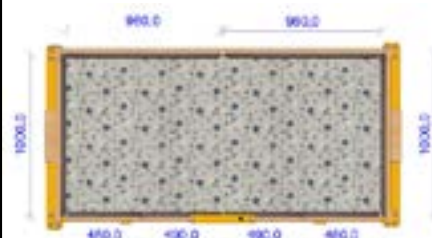
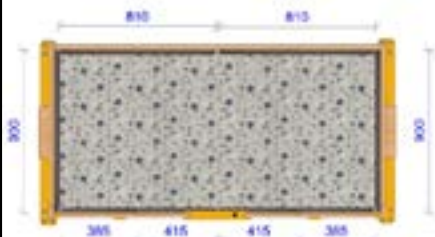
## Operating instruction for our skai-padding

Articel-No.: 99.051

(lying area 90x170cm)


Contains:

- 2 padding 810 x 500 mm (BxH)
- 2 padding 900 x 500 mm (BxH)
- 2 padding 385 x 500 mm (BxH)
- 2 padding 415 x 500 mm (BxH)

B = width

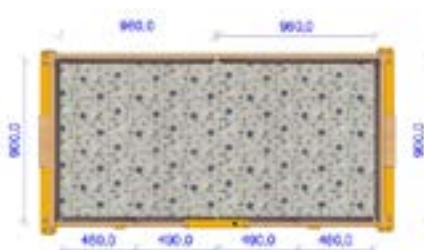(lying area 90x200cm)


Contains:

- 2 padding 960 x 500 mm (BxH)
- 2 padding 900 x 500 mm (BxH)
- 2 padding 460 x 500 mm (BxH)
- 2 padding 490 x 500 mm (BxH)

H = Height

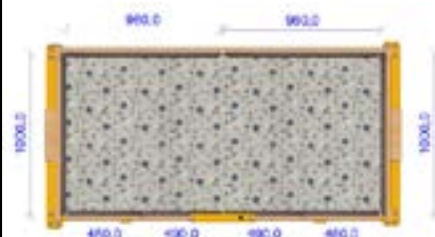(lying area 100x200cm)

Contains:

- 2 padding 960 x 500 mm (BxH)
- 2 padding 1000 x 500 mm (BxH)
- 2 padding 460 x 500 mm (BxH)
- 2 padding 490 x 500 mm (BxH)

Please take the arrangement of skai-padding as is discribed above.

You can fix the skai-padding individually because there are 3 diffent versions.

### NOTE:

Before the doors of the children's care bed can be opened, you must remove the skai-padding in this area. Also, the electric height adjustment can be used only if the skai-padding have been previously removed.

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#### Cleaning and care:

Artificial leather has softener and malleable making additives that provide it with a supple property.

A professional and competent cleaning is essential to maintain the colours and to prevent material hardening.

For lightly soiled material we suggest cleaning with a soft cloth and soapsuds.

Single spots (for example ballpen ink) have to be treated immediately with alcoholic detergent with at least 50 % alcohol.

Never use aggressive chemical detergents or abrasives!

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[www.savi.de](http://www.savi.de)  
[info@savi.de](mailto:info@savi.de)

SAVI GmbH  
Reha Solutions  
Lise-Meitner-Str. 10  
D-48691 Vreden

Postfach 1123  
D-48684 Vreden  
Fon: +49 (0) 25 64 - 93 09-70  
Fax: +49 (0) 25 64 - 93 09-60