

# Tom

Curious how the child care bed Tom (90 x 170 cm, 113 cm barred side height) looks in your nursery? →



Simply scan the QR code and place the bed visually in your nursery!



**Great colour?**  
Available for every one of our beds and other wood colours! See page 79.

← e.g. in wood colour 'Wenge'

Special beds for unique children – our Tom can be designed variably according to the needs of your child and therefore always offers the right solution.

Thanks to the four-winged folding sliding door, the barred sides and the flexible height adjustment possibilities, our Tom provides an optimal therapeutic support. Due to the low entry height of a little about 20 cm even smaller children can get into the bed on their own.

low entry height of about 20 cm possible



Electric height adjustment



Head- & footelement manual or electric adjustment (option)

Discover our improved corner posts in the ball chain test!



Scan the QR code and find out more about our Improved corner posts (for all our beds)!



More Info about Tom:  
[www.savi.de/env/care-beds-for-children/](http://www.savi.de/env/care-beds-for-children/)

EN 60601-2-52  
DIN EN 50637

According to Medical Device Regulation  
CE-Markering  
Declaration of Conformity



# Tom

Best security for your child,  
thanks to optimal locking systems

## Why a bed from SAVI?

- Great safety for your child
- Biologically neutral, pollutant-free (DIN EN 71-3)
- Stability and reliability
- Comfortable, back-friendly height electric
- Many bed sizes
- Individual equipment options
- Optimal eye contact with your child
- Large, safe indoor area
- Colourful padding for maximum protection
- Short delivery times

## Tom's individual strengths

- Four-winged folding sliding door with comfort handle
- Castors
- High barred sides (option)
- Low entry height of about 20 cm (option)
- Trendelenburg- / Antitrendelenburg-function (option)
- Fixed Skai-padding (also with vinyl windows) (option)



Choice of colours (see on page 78)

A lot of safety, even for  
particularly active  
children



Barred side height 72 cm,  
e.g. In blue-purple colour accent with wooden castors



Barred side height 113 cm,  
e.g. In blue-purple colour accent with wooden castors



Barred side height 135 cm,  
e.g. In blue-purple colour accent with wooden castors

## Technical data

Lying area	External dimension	Article-no.
100 x 200 cm	115,5 x 223 cm	155.34.008
90 x 200 cm	105,5 x 223 cm	155.34.008.L
90 x 170 cm	105,5 x 193 cm	155.34.008.M
65 x 155 cm	80,5 x 178 cm	155.34.008.S
70 x 140 cm	85,5 x 163 cm	155.34.008.XS
Doors		
Four-winged folding sliding door		•
Four-winged folding sliding door on both sides	99.750	+
Folding door in the 270° version	99.704	+
Folding door in the 270° version on both sides	99.704BEID	+
Locking mechanism		
Child-proof SAVI-lock I		•
Child-proof SAVI-lock I metal	40100003	+
SAVI-lock II	99.033	+
SAVI-lock III	99.034	+
SAVI-lock IV	99.034A	+
Lying height (top of slatted bed base)		
34 - 104 cm electric height adjustment		•
Barred side height		
	Overall height	
72 cm	114 cm	+
113 cm	155 cm	•
135 cm	177 cm	+
Special sizes on request		+
Slatted bed base		
No adjustment (fixed)		+
Manual adjustment	99.030	•
Electric adjustment	99.200	+
Trendelenburg-/Antitrendelenburg-funktion (tilting of the entire bed)	99.211	+
Running gear		
Four castor running gear with total locking mechanism		•
Designer wooden castors	99.885	n.a.c.
Without castors and low entrance height of 20 cm	99.883	+
Small castors and low entrance height of 27 cm	99.881	+
Kind of wood		
Beechwood „natural oiled“ (further colours see p. 79)		•
Mattresses		
Polyfoam mattress	99.031.4	•
Cold foam mattress	99.031.5	+
Reha-star mattress plus (up to stage 2)	99.031.3.PLUS	+
Reha-star mattress II (up to stage 4)	99.031.3.ZWEI	+
Incontinence cover	99.031.3ZA50	+

• = standard    o.r. = on request    + = accessories    n.a.c. = no additional charge

## Technical data

Coloured features		
Natural	90.100	n.a.c.
Yellow	90.128	n.a.c.
Orange	90.204	n.a.c.
Blue-purple	90.405	•
Purple	90.406	n.a.c.
Blue	90.515	n.a.c.
Green (for more colours see page 78)	90.618	n.a.c.
Equipment		
Children's name on side of bed	Kindemame	n.a.c.
Coloured sticks		n.a.c.
Perspex instead of sticks		•
Sticks instead of perspex	99.815	n.a.c.
Touch protection for height adjustment	99.900	+
External lock on side of bed	99.706	+
Tube feed-through	99.840	o.r.
Raising aid	99.802	+
Net-roof	99.800	+
Adaption of the bed for sloping ceiling		o.r.
Mattress holder	99.891	+
Protection for the bed base in case of kicks	99.892	+
Wood care set	99.880	+
Padding		
Skai-padding 50 cm height	99.051	+
Skai-padding 100 cm height	99.051.100	+
Fixed Skai-padding	99.051.FEST	o.r.
Fixed Skai-padding with vinyl windows	99.051.FEST	o.r.
Edge padding	99.053	+

• = standard    o.r. = on request    + = accessories    n.a.c. = no additional charge

Our Skai-padding is in many colours available!

You can find a pre-selection on our website at:

[www.savi.de/en/bed-padding/](http://www.savi.de/en/bed-padding/)





Tom


## Open side front thanks to four-winged folding sliding door

Thanks to its folding sliding mechanism, the door offers the maximum stability and precise operation in every position. The design is a harmonious combination of quality and function. The entire front side can be opened in this way.

Tom adapts perfectly to the situation and makes everyday life with your child easier. Naturally by request we can also offer customised additional doors, and custom-made sloping roofs are also not a problem for us.

## Safety first – SAVI door locks

- 1 Child-proof SAVI-lock I (base)
- 2 Child-proof SAVI-lock I made of metal
- 3 SAVI-lock II
- 4 SAVI-lock III: The labyrinth guide prevents the child opening the bed by accident. This lock is keyless and has no magnets. Can also be retrofitted.
- 5 SAVI-lock IV: Here too the bed is opened by means of a labyrinth guide, but in addition this lock also operates against resistance. This lock is keyless, without magnets and can be retrofitted.
- 6 External lock on side of bed (increases the stability and provides more secure)

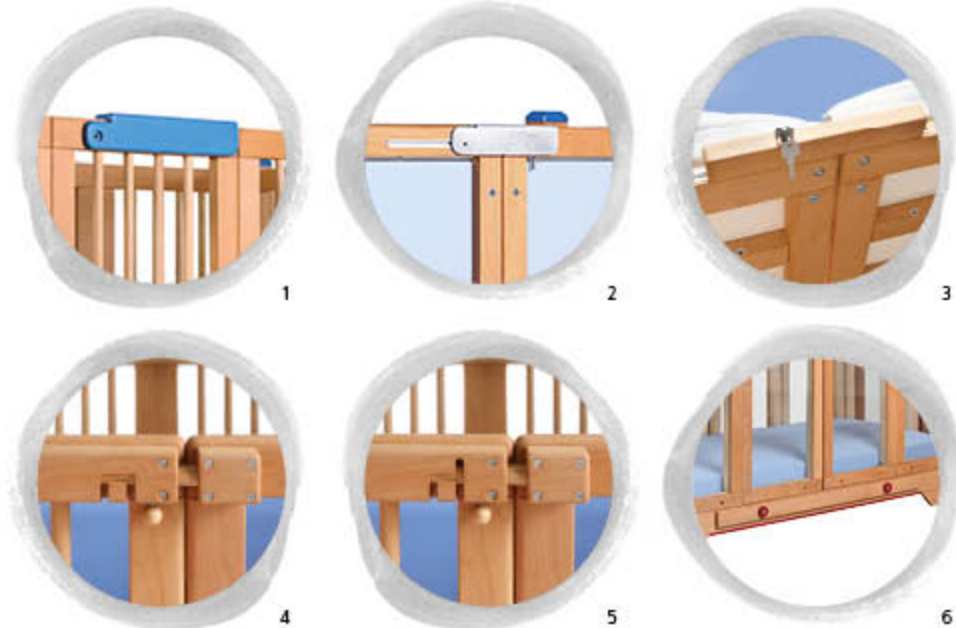

Great solutions for  
door locks – highest  
safety first

## Key data of Tom

- Barred side height 72 cm,  
Barred side height 113 cm,  
Barred side height 135 cm,  
special sizes on request
- Lying height 34 - 104 cm or  
Lying height 20 - 90 cm (without castors)
- Trendelenburg-/Antitrendelenburg-function (option)
- Choose between perspex or sticks

## 270° door to fold back completely

- Easy to use manually
- Doors on the head and foot elements fold back completely
- 270° version can also be supplied with fixed Skai-padding



ID: 35656



ID: 20009





Tom

All functions of the bed can always be used without removing the padding beforehand!

## Optimum protection thanks to fixed Skai-padding

Your child is very active? With the fixed Skai-padding (75 mm thickness), you offer your child optimum protection.

For more information on the different padding options, see page 79/ 80!

## Optimal view thanks to integrated vinyl windows

The integrated vinyl windows fit flush with the fixed Skai-padding and thus offers a similar padding effect to the permanently integrated foam padding.

The windows offer an optimal view. So you keep an eye on your child easily and your child feels visibly more comfortable.



ID: 36009  
Supply opening (hole)  
with closing mechanism



ID: 36449  
Supply opening (hole)  
with closing mechanism



Scan the QR code to find out more about our fixed Skai-padding with vinyl windows and its padding capability.



ID: 37975  
External door lock

# Examples



ID: 38202  
Colour accent grey,  
lockable Handset-box,  
external lock



ID: 36422  
Child's name on the side  
of the bed



ID: 35057  
Low entrance height  
(without castors)



ID: 31270  
White lacquered  
(opaque)



ID: 34948  
Without castors,  
coloured sticks



ID: 37188  
Step protection for  
the slatted frame




ID: 34204  
Sticks, wooden castors and  
270° door on both sides



ID: 34923  
Touch protection  
for height adjustment

„Sooo much space –  
I'd love to cuddle  
in it too!“

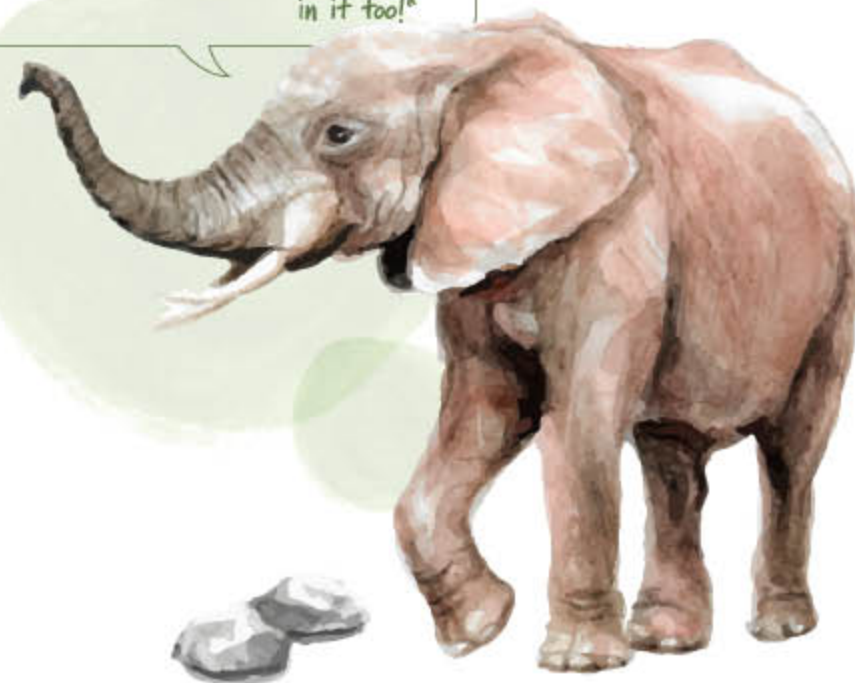



# Tom Paddy

## Tom Paddy's individual strengths

- Low entry height of 20 cm minimum
- High barred sides 113 cm
- Fixed Skai-padding with vinyl windows
- Castors (option)
- Trendelenburg-/Antitrendelenburg-function (option)



← low entry height of about 20 cm possible



## Technical data

Lying area	External dimension	Article-no.
90 x 200 cm	120 x 238 cm	155.34.008.L PADDY
90 x 170 cm	120 x 208 cm	155.34.008.M PADDY
<b>Doors</b>		
Four-winged folding sliding door		•
Four-winged folding sliding door on both sides	99.750	+
Folding door in the 270° version	99.704	+
Folding door in the 270° version on both sides	99.704BEID	+
<b>Locking mechanism</b>		
Child-proof SAVI-lock I		•
Child-proof SAVI-lock I metal	40100003	+
SAVI-lock II	99.033	+
SAVI-lock III	99.034	+
SAVI-lock IV	99.034A	+
<b>Lying height (top of slatted bed base)</b>		
20 - 90 cm electric height adjustment (without castors)		•
<b>Door height</b>	<b>Overall height</b>	
113 cm	149 cm	+
<b>Slatted bed base</b>		
Manual adjustment		•
Electric adjustment	99.200	+
Trendelenburg-/Antitrendelenburg-function (tilting of the entire bed)	99.211	+
<b>Running gear</b>		
Without castors and low entrance height of 20 cm	99.883	+
Small castors and low entrance height of 27 cm	99.881	+
<b>Kind of wood</b>		
Beechwood „natural oiled“ (further colours see p. 79)		•
<b>Mattresses</b>		
Polyfoam mattress	99.031.4	•
Cold foam mattress	99.031.5	+
Reha-star mattress plus (up to stage 2)	99.031.3.PLUS	+
Reha-star mattress II (up to stage 4)	99.031.3.ZWEI	+
Incontinence cover	99.031.3ZA50	+

• = standard    o.r. = on request    + = accessories

## Technical data

<b>Equipment</b>		
External lock on side of bed	99.706	+
Raising aid	99.802	+
Net-roof	99.800	+
Mattress holder	99.891	+
Protection for the bed base in case of kicks	99.892	+
Wood care set	99.880	+
<b>Padding</b>		
Fixed Skal-padding with vinyl windows	99.051.FEST	•

• = standard    o.r. = on request    + = accessories

Fixed padding in creme or light-blue.





# Create your own



## Create your children's bed for rehabilitation!

This is a list which you can use to create your children's bed for rehabilitations. We will take your wishes into consideration and make an offer about one of our bed's.

Just make a cross and decide how your bed should look like:

### 1. Lying area of the bed?

- 65 x 155 cm    70 x 140 cm    90 x 170 cm  
 90 x 200 cm    100 x 200 cm   other size: \_\_\_\_\_

### 2. Which barred side do you need? Height (cm): \_\_\_\_\_

### 3. Do you use a patient-lifter? Yes No

### 4. Which lying height (top of slatted base) do you need?

- 40 cm    50 cm    60 cm    70 cm   other height: \_\_\_\_\_

### 5. Do you need an electric height adjustment? Yes No

### 6. Which slatted bed base do you need?

- no adjustment    manual adjustment    step-protection for the bed base  
 electric adjustment    Trendelenburg-function

### 7. What kind of doors do you want?

- 4-winged folding sliding door    270° door    side rail "hide away"    folding door  
 lowerable side rail    electric lowerable side rail

### 8. How many doors do you need?

- front side    front and back sides    head and footelement

### 9. Locking mechanism?

- SAVI-lock I    SAVI-lock II    SAVI-lock III    SAVI-lock IV

### 10. Do you need perspex?

- Yes. Where?: \_\_\_\_\_    No

### 11. Kind of wood / wood-colour? Wood:

- beech    pine  
Wood-colour:  natural    basalt    wenge    barrique    teak    walnut  
 white stained    white lacquered

### 12. Colour feature?

- natural    yellow    orange    red    grey    blue-purple    purple  
 blue    green    white    pink    other colour: \_\_\_\_\_  
 coloured sticks    children's name: \_\_\_\_\_

### 13. Running gear? castors   without castors

### 14. Do you want drawer? Yes No

### 15. Do you want a mattress? Yes No

If yes, which one?

- polyfoam mattress    interior spring mattress    cold foam mattress  
 Reha-Star II    Reha-Star Plus    incontinence cover

### 16. Do you need a padding? Yes No

If yes, which one?

- loose Skai-padding    fixed Skai-padding    edge-padding    motifpadding MALU  
height of the padding: \_\_\_\_\_ colour of the padding: \_\_\_\_\_

### 17. Do you need Vinyl-windows?

- Yes. Where? \_\_\_\_\_    No

### 18. Equipment?

- removable posts (Niklas)    net-roof    raising aid    lockable handset box  
 tube feed-through (gap)    infusion holder    drawers    mattress holder  
 touch-protection for height adjustment    foot cut-out (e.g. for patient lift)  
 central locking device for 4-roller chassis (Knut a+b only)    access aid made of artificial leather  
 supply opening (hole) with closing mechanism

Please use this statements to create an offer, where you choose a model, which fulfils the handicaps to the greatest possible extent.

Please use this statements to create an offer about a special model to fulfil the handicaps exactly.

place, date

stamp, sign

**Very easy:**

Just send us your order via email

or use our online-form at: [www.savi.de/en/home/](http://www.savi.de/en/home/)

SAVI GmbH Reha Solutions

Lise-Meitner-Str. 10   Fon: +49 (0) 25 64 - 93 09-70

D-48691 Vreden   Fax: +49 (0) 25 64 - 93 09-60